

OPSITE SPRAY



MAIN SPECIFICATIONS

- Opsite Spray is a transparent and quick drying film which is easy and convenient to apply. The film helps to provide protection for dry minor surgical and surface wounds. Forms a transparent film that dries quickly, the film remains intact even in areas of joints that are in frequent motion, protecting against bacterial infections.
- Permeable to moisture vapour and air.
- Water resistant.
- Conformable and elastic.
- Quick and easy to apply.
- This product contains no Chlorofluorocarbon (CFC) propellants.
- Permeable to moisture vapour and air - Opsite Spray is permeable to moisture vapour and air, allowing the surrounding skin to breathe, so preventing maceration.
- Water resistant - The skin area can be bathed easily without affecting the dressing or the wound.
- Conformable and elastic - The film stays intact over skin areas that are flexible or mobile such as knuckles.
- Quick and Easy to apply.
- Opsite Spray can be used for a variety of minor dry wounds, in particular: Minor cuts and abrasions; Over

dry sutured wounds; Sealing of drainage tubes and external orthopaedic fixation, such as Steinman pins; After suture removal; Over vaccination sites; Skin graft fixation; Can help to protect unbroken blisters; Can help to protect the skin from friction and so help prevent blisters.

- Avoid inhaling.
- Use in a well ventilated area.
- Avoid contact with eyes.
- Do not spray on a naked flame or any incandescent material.
- Pressurised container: protect from sunlight and do not expose to temperatures exceeding 50 degrees Celsius. Do not pierce or burn, even after use.
- Keep out of the reach of children.
- For external use only.
- Keep away from sources of ignition - No smoking.
- Do not use with plastic intravascular devices.
- Avoid spraying onto fabrics.
- Not to be used with iodine preparations. Skin reddening may occur when used with an iodine solution and covered with occlusive dressings.
- Not intended to be used in conjunction with other medicated preparations.
- Do not apply on children under the age of 18 months.
- For skin grafts: It is imperative that adequate areas of the graft are left without spray to allow adequate drainage of serous fluid.

Code	Description	Selling unit
01003008	OPSITE SPRAY 240 ML	CX. 1