

## VARIHESIVE HYDROGEL



### MAIN SPECIFICATIONS

- Hydrogel tube with applicator.
- Sterile, composed with natural hydrocolloid: pectin, sodium carboxymethylcellulose presented in a transparent viscous vehicle.
- It is indicated in vascular ulcers and pressure ulcers.
- The gel creates an ideal moisture atmosphere for healing, favoring natural autolytic debridement process.
- Behaves according to the level of wound hydration promoting their hydrobalance.
- Usage directions: Clean the wound with distilled water, saline, or with soap and water. Avoid the use of already antiseptic solutions which slow the growth of granulation tissue. Carefully dry the skin around the lesion. Unscrew the tube cap, remove the retaining ring and flip the cover to punch him tube. Apply the gel directly into the lesion without overlap in the skin that surrounds it. Use a secondary dressing to cover the injury. You should change the dressing when there is a gel leakage of secondary dressing, or according to routine treatment. Remove the gel remains with saline. In any case it should be a maximum change dressing every 3 days, and when used on necrotic lesions, for cleaned or injuries granulation process it is recommended every 7 days.
- Contraindicated in patients with known sensitivity to dressing and its components.
- Does not contain latex.

| Code     | Description                | Selling unit |
|----------|----------------------------|--------------|
| 01101015 | VARIHESIVE HYDROGEL 15 GRS | CX. 10       |
| 01101000 | VARIHESIVE HYDROGEL 15 GRS | CX. 5        |