

# VARIHESIVE PASTE OINTMENT HYDROACTIVE



## MAIN SPECIFICATIONS

- Sterile ointment / paste to fill wounds with mild and moderately exuding. Contains natural hydrocolloid fat of hypoallergenic base.
- Contraindicated in patients with known sensitivity to product or their components.
- Indicated for the treatment of ulcers and lesions with abundant exudation. Its purpose is to absorb excess exudate from lesions by increasing the residence time of the used dressing and reducing the frequency of change. Another purpose is to fill irregularities or recesses of the wound bed, and thus facilitate the execution of another bandage or gel plate. Suitable for wounds where a conventional dressing does not make an effective coverage of the wound bed and wound with difficult access areas.
- The contents of the tube is sterile until they are opened, so it should be used for an application.
- Instructions for use: Clean the area of the lesion, rinse well and dry the surrounding skin. Spread the paste on the wound bed cavities with a sterile spatula. The folder should not exceed the edge of the lesion. Making a simple washing with enough saline to remove gel damage when the need to replace the dressing.
- Does not contain latex.

Code	Description	Selling unit
01101016	VARIHESIVE PASTE 30 GRS	CX. 1