

ASKINA TOUCH HIDROCELULAR DRESSING



MAIN SPECIFICATIONS

- Sterile, non-adhesive polyurethane foam dressing.
- Multi-layered structure with high absorption capacity, hydrogel sheet ensures maximum absorption and retention of the exudate, while the polyurethane outer layer prevents the penetration of liquids and other contaminants, allowing gas exchange.
- Requires a secondary fixation for fixation.
- Askina Touch is indicated for the treatment of a series of variable depth lesions, such as decubitus ulcers, arterial and venous ulcers, ulcers or diabetic foot ulcers.
- Askina Touch is easy to apply without requiring special equipment.
- Askina Touch can be left in place for several days, depending on the amount of exudate.
- Clean the wound with a cleansing and decontamination solution and dry the skin around the lesion.
- Select the appropriate size of Askina Touch completely covering the surface of the lesion, ensuring a margin of 2 to 3 cm around the edges of the lesion.
- If necessary, Askina Touch can be cut to size before removing the protective layers. Where necessary, it is possible to overlap several patches to cover large lesions.

- Remove finishes. Put Askina touch the damage and gently squeeze to ensure proper application.
- Askina Touch can be left in place for several days to a week, depending on the condition of the injury.

Code	Description	Selling unit
7261002	ASKINA TOUCH 10X10CM	CX. 10
7261501	ASKINA TOUCH 10X15CM	CX. 5
7262253	ASKINA TOUCH 15X15CM	CX. 10
7264001	ASKINA TOUCH 20X20CM	CX. 5